

MIC Test Strip

for antimicrobial susceptibility testing



Quantitative assay for determining the Minimum Inhibitory Concentration (M.I.C.)


MIC Test Strip is a quantitative assay for determining the Minimum Inhibitory Concentration (M.I.C.) of antimicrobial agents against microorganisms and for detecting the resistance mechanisms.

MIC Test Strip are porous strips with special features (**Italian Patent no. 1395483 and International Patent pending**) that are impregnated with a predefined concentration gradient of antibiotic, across 15 two-fold dilutions of a conventional M.I.C. method.

On one side of the strip is indicated a M.I.C. scale in $\mu\text{g/mL}$ and a code that identify the antimicrobial agent.

For ESBL, MBL, GRD, AmpC and KPC detection, the double-sided gradient carries the appropriate diagnostic reagents.

MIC Test Strip are available in a large variety of configurations. Each configuration is available in packages of 10, 30 and 100 tests.

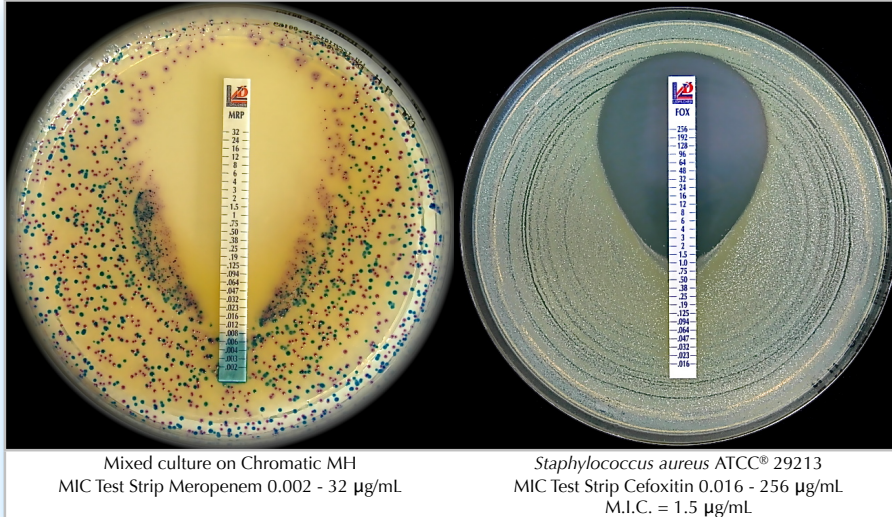


Mixed culture on Chromatic MH
MIC Test Strip Meropenem 0.002 - 32 $\mu\text{g/mL}$

Staphylococcus aureus ATCC® 29213
MIC Test Strip Cefoxitin 0.016 - 256 $\mu\text{g/mL}$
M.I.C. = 1.5 $\mu\text{g/mL}$

method principle

When the **MIC Test Strip** is applied onto an inoculated agar surface, the preformed exponential gradient of antimicrobial agent is immediately transferred to the agar matrix. After 18 hours incubation or longer, a symmetrical inhibition ellipse centered along the strip is formed. The MIC is read directly from the scale in terms of $\mu\text{g/mL}$ at the point where the edge of the inhibition ellipse intersects the strip **MIC Test Strip**.

Other growth/inhibition patterns may also be seen for resistance detection methods.

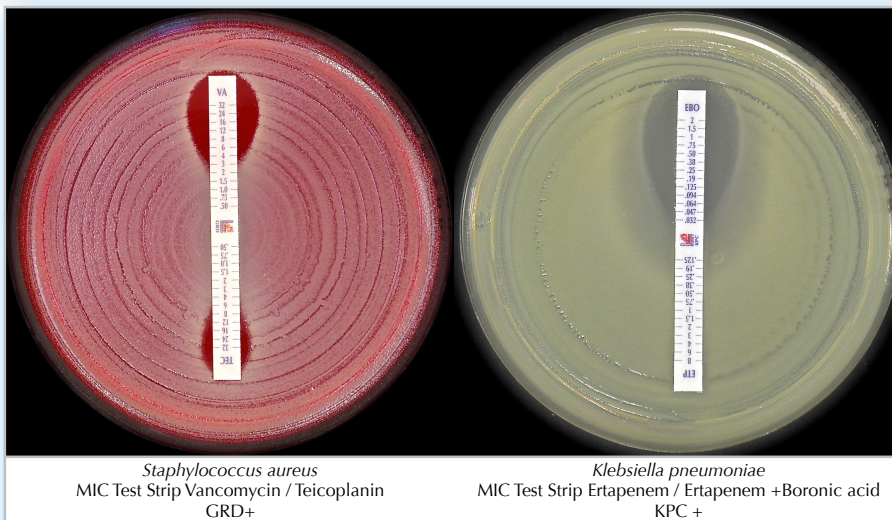


Staphylococcus aureus
MIC Test Strip Vancomycin / Teicoplanin
GRD+

Klebsiella pneumoniae
MIC Test Strip Ertapenem / Ertapenem + Boronic acid
KPC +

BIBLIOGRAPHY

- CLSI M100-S24, 2014. Performance Standards for Antimicrobial Susceptibility Testing.
- CLSI M7-A9, 2012. Methods for Dilution Antimicrobial Susceptibility Tests for Bacteria that Grow Aerobically.
- CLSI M11-A8 Methods for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M11-S1 Performance Standards for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M27-A3. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Approved Standard - Third Edition.
- CLSI M27-S4. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Fourth Informational supplement.
- EUCAST. Breakpoint tables for interpretation of MICs and zone diameters Version 4.0, January 2014.

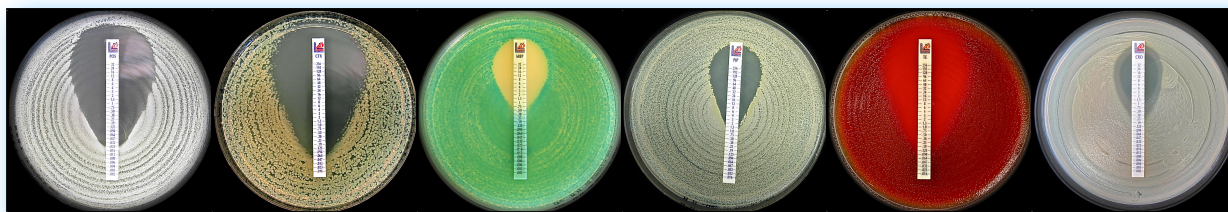

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		$\mu\text{g/mL}$	Code	Packaging	Ref.
MIC Test Strip	AMIKACIN	0.016 - 256	AK	10 strips 30 strips 100 strips	920181 92018 920180
MIC Test Strip	AMOXICILLIN	0.016 - 256	AML	10 strips 30 strips 100 strips	920211 92021 920210
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2/1) <i>CLSI recommended</i>	0.016 - 256*	AUG	10 strips 30 strips 100 strips	920241 92024 920240
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2 $\mu\text{g/mL}$) <i>EUCAST recommended</i>	0.016 - 256*	AMC	10 strips 30 strips 100 strips	921801 92180 921800
MIC Test Strip	AMPICILLIN	0.016 - 256	AMP	10 strips 30 strips 100 strips	920031 92003 920030
MIC Test Strip	AMPICILLIN*-SULBACTAM (2/1) <i>CLSI recommended</i>	0.016 - 256*	AMS	10 strips 30 strips 100 strips	920271 92027 920270
MIC Test Strip	AMPICILLIN*-SULBACTAM (4 $\mu\text{g/mL}$) <i>EUCAST recommended</i>	0.016 - 256*	SAM	10 strips 30 strips 100 strips	921811 92181 921810
MIC Test Strip	AZITHROMYCIN	0.016 - 256	AZM	10 strips 30 strips 100 strips	920301 92030 920300
MIC Test Strip	AZTREONAM	0.016 - 256	ATM	10 strips 30 strips 100 strips	920331 92033 920330
MIC Test Strip	AZTREONAM	0.064 - 1024	ATM	10 strips 30 strips 100 strips	921731 92173 921730
MIC Test Strip	BACITRACIN	0.016 - 256	BA	10 strips 30 strips 100 strips	920191 92019 920190
MIC Test Strip	CEFACLOR	0.016 - 256	CEC	10 strips 30 strips 100 strips	920361 92036 920360
MIC Test Strip	CEFAZOLIN	0.016 - 256	KZ	10 strips 30 strips 100 strips	921741 92174 921740
MIC Test Strip	CEFEPIME	0.016 - 256	FEP	10 strips 30 strips 100 strips	921261 92126 921260
MIC Test Strip	CEFIXIME	0.016 - 256	CFM	10 strips 30 strips 100 strips	920601 92060 920600
MIC Test Strip	CEFOPERAZONE*-SULBACTAM (2/1)	0.016 - 256*	CPS	10 strips 30 strips 100 strips	920231 92023 920230
MIC Test Strip	CEFOTAXIME	0.002 - 32	CTX	10 strips 30 strips 100 strips	920071 92007 920070
MIC Test Strip	CEFOTAXIME	0.016 - 256	CTX	10 strips 30 strips 100 strips	920061 92006 920060
MIC Test Strip	CEFOTETAN	0.016 - 256	CTT	10 strips 30 strips 100 strips	920201 92020 920200
MIC Test Strip	CEFOXITIN	0.016 - 256	FOX	10 strips 30 strips 100 strips	920661 92066 920660
MIC Test Strip	CEFPIROME	0.016 - 256	CR	10 strips 30 strips 100 strips	920081 92008 920080
MIC Test Strip	CEFPODOXIME	0.016 - 256	PX	10 strips 30 strips 100 strips	920051 92005 920050
MIC Test Strip	CEFTAROLINE	0.016 - 256	CPT	10 strips 30 strips 100 strips	920491 92049 920490
MIC Test Strip	CEFTAZIDIME	0.016 - 256	CAZ	10 strips 30 strips 100 strips	921381 92138 921380
MIC Test Strip	CEFTIBUTEN	0.002 - 32	CTB	10 strips 30 strips 100 strips	920581 92058 920580
MIC Test Strip	CEFTIZOXIME	0.002 - 32	CZX	10 strips 30 strips 100 strips	920161 92016 920160
MIC Test Strip	CEFTRIAXONE	0.016 - 256	CRO	10 strips 30 strips 100 strips	920421 92042 920420

MIC Test Strip for ANTIMICROBIAL susceptibility testing

Description	µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFTRIAZONE	CRO	10 strips 30 strips 100 strips	920431 92043 920430
MIC Test Strip	CEFUROXIME	CXM	10 strips 30 strips 100 strips	921291 92129 921290
MIC Test Strip	CEPHALOTHIN	KF	10 strips 30 strips 100 strips	920391 92039 920390
MIC Test Strip	CHLORAMPHENICOL	C	10 strips 30 strips 100 strips	920751 92075 920750
MIC Test Strip	CIPROFLOXACIN	CIP	10 strips 30 strips 100 strips	920451 92045 920450
MIC Test Strip	CLARITHROMYCIN	CLR	10 strips 30 strips 100 strips	920481 92048 920480
MIC Test Strip	CLINDAMYCIN	CD	10 strips 30 strips 100 strips	920721 92072 920720
MIC Test Strip	CLOXACILLIN	CX	10 strips 30 strips 100 strips	920441 92044 920440
MIC Test Strip	COLISTIN	CS	10 strips 30 strips 100 strips	921411 92141 921410
MIC Test Strip	COLISTIN	CS	10 strips 30 strips 100 strips	921421 92142 921420
MIC Test Strip	DAPTOMYCIN (Includes Ca ²⁺)	DAP	10 strips 30 strips 100 strips	921451 92145 921450
MIC Test Strip	DORIPENEM	DOR	10 strips 30 strips 100 strips	920401 92040 920400
MIC Test Strip	DOXYCYCLINE	DXT	10 strips 30 strips 100 strips	921561 92156 921560
MIC Test Strip	ENROFLOXACIN	ENR	10 strips 30 strips 100 strips	920131 92013 920130
MIC Test Strip	ERTAPENEM	ETP	10 strips 30 strips 100 strips	921571 92157 921570
MIC Test Strip	ERYTHROMYCIN	E	10 strips 30 strips 100 strips	920511 92051 920510
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	FOS	10 strips 30 strips 100 strips	920781 92078 920780
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	FOS	10 strips 30 strips 100 strips	920791 92079 920790
MIC Test Strip	FUSIDIC ACID	FU	10 strips 30 strips 100 strips	920021 92002 920020
MIC Test Strip	GATIFLOXACIN	GAT	10 strips 30 strips 100 strips	920111 92011 920110
MIC Test Strip	GEMIFLOXACIN	GEM	10 strips 30 strips 100 strips	920351 92035 920350
MIC Test Strip	GENTAMICIN	CN	10 strips 30 strips 100 strips	920091 92009 920090
MIC Test Strip	GENTAMICIN	CN	10 strips 30 strips 100 strips	920101 92010 920100
MIC Test Strip	IMPENEM	IMI	10 strips 30 strips 100 strips	920541 92054 920540
MIC Test Strip	KANAMYCIN	K	10 strips 30 strips 100 strips	920341 92034 920340
MIC Test Strip	LEVOFLOXACIN	LEV	10 strips 30 strips 100 strips	920811 92081 920810
MIC Test Strip	LINEZOLID	LNZ	10 strips 30 strips 100 strips	921351 92135 921350
MIC Test Strip	MECILLINAM	MEC	10 strips 30 strips 100 strips	920171 92017 920170
MIC Test Strip	MEROPENEM	MRP	10 strips 30 strips 100 strips	920841 92084 920840
MIC Test Strip	METRONIDAZOLE	MTZ	10 strips 30 strips 100 strips	920871 92087 920870

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MOXIFLOXACIN	0.002 - 32	MXF	10 strips 30 strips 100 strips	920901 920900 920900
MIC Test Strip	MUPIROCIIN	0.064 - 1024	MUP	10 strips 30 strips 100 strips	920381 920380 920380
MIC Test Strip	MINOCYCLINE	0.016 - 256	MN	10 strips 30 strips 100 strips	920321 920320 920320
MIC Test Strip	NALIDIXIC ACID	0.016 - 256	NA	10 strips 30 strips 100 strips	921321 921320 921320
MIC Test Strip	NETILMICIN	0.016 - 256	NET	10 strips 30 strips 100 strips	920931 920930 920930
MIC Test Strip	NORFLOXACIN	0.016 - 256	NOR	10 strips 30 strips 100 strips	920961 920960 920960
MIC Test Strip	NITROFURANTOIN	0.032 - 512	F	10 strips 30 strips 100 strips	920221 920220 920220
MIC Test Strip	OFLOXACIN	0.002 - 32	OFX	10 strips 30 strips 100 strips	920991 920990 920990
MIC Test Strip	OXACILLIN	0.016 - 256	OX	10 strips 30 strips 100 strips	920151 920150 920150
MIC Test Strip	PENICILLIN G	0.002 - 32	P	10 strips 30 strips 100 strips	921031 921030 921030
MIC Test Strip	PENICILLIN G	0.016 - 256	P	10 strips 30 strips 100 strips	921021 921020 921020
MIC Test Strip	PIPERACILLIN	0.016 - 256	PIP	10 strips 30 strips 100 strips	921051 921050 921050
MIC Test Strip	PIPERACILLIN*-TAZOBACTAM (4 µg/mL)	0.016 - 256*	TZP	10 strips 30 strips 100 strips	921081 921080 921080
MIC Test Strip	POLYMYXIN B	0.064 - 1024	PB	10 strips 30 strips 100 strips	920041 920040 920040
MIC Test Strip	QUINUPRISTIN-DALFOPRISTIN	0.002 - 32	QDA	10 strips 30 strips 100 strips	920261 920260 920260
MIC Test Strip	RIFAMPICIN	0.002 - 32	RD	10 strips 30 strips 100 strips	920011 920010 920010
MIC Test Strip	RIFAMPICIN	0.016 - 256	RD	10 strips 30 strips 100 strips	920251 920250 920250
MIC Test Strip	SPECTINOMYCIN	0.064 - 1024	SPC	10 strips 30 strips 100 strips	920141 920140 920140
MIC Test Strip	SPIRAMYCIN	0.002 - 32	SP	10 strips 30 strips 100 strips	920461 920460 920460
MIC Test Strip	STREPTOMYCIN	0.064 - 1024	S	10 strips 30 strips 100 strips	921111 921110 921110
MIC Test Strip	SULBACTAM	0.016 - 256	SUL	10 strips 30 strips 100 strips	920281 920280 920280
MIC Test Strip	SULFAMETHOXAZOLE	0.064 - 1024	SMX	10 strips 30 strips 100 strips	920311 920310 920310
MIC Test Strip	TEICOPLANIN	0.016 - 256	TEC	10 strips 30 strips 100 strips	920121 920120 920120
MIC Test Strip	TELAVANCIN	0.002 - 32	TLV	10 strips 30 strips 100 strips	920521 920520 920520
MIC Test Strip	TELAVANCIN	0.016 - 256	TLV	10 strips 30 strips 100 strips	920531 920530 920530
MIC Test Strip	TEMOCILLIN	0.064 - 1024	TMO	10 strips 30 strips 100 strips	920291 920290 920290
MIC Test Strip	TETRACYCLINE	0.016 - 256	TE	10 strips 30 strips 100 strips	921141 921140 921140
MIC Test Strip	TIAMULIN	0.002 - 32	TIA	10 strips 30 strips 100 strips	922001 922000 922000
MIC Test Strip	TICARCILLIN*-CLAVULANIC ACID (2 µg/mL)	0.016 - 256*	TTC	10 strips 30 strips 100 strips	921171 921170 921170
MIC Test Strip	TIGECYCLINE	0.016 - 256	TGC	10 strips 30 strips 100 strips	921441 921440 921440

MIC Test Strip for ANTIMICROBIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	TILMICOSIN	0.002 - 32	TIL	10 strips 30 strips 100 strips	922011 92201 922010
MIC Test Strip	TOBRAMYCIN	0.016 - 256	TOB	10 strips 30 strips 100 strips	921211 92121 921210
MIC Test Strip	TOBRAMYCIN	0.064 - 1024	TOB	10 strips 30 strips 100 strips	921201 92120 921200
MIC Test Strip	TRIMETHOPRIM	0.002 - 32	TM	10 strips 30 strips 100 strips	920371 92037 920370
MIC Test Strip	TRIMETHOPRIM*-SULFAMETHOXAZOLE (1/19)	0.002 - 32*	SXT	10 strips 30 strips 100 strips	921231 92123 921230
MIC Test Strip	VANCOMYCIN	0.016 - 256	VA	10 strips 30 strips 100 strips	920571 92057 920570

MIC Test Strip for ANTIFUNGAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	AMPHOTERICIN B	0.002 - 32	AMB	10 strips 30 strips 100 strips	921531 92153 921530
MIC Test Strip	ANIDULAFUNGIN	0.002 - 32	AND	10 strips 30 strips 100 strips	921551 92155 921550
MIC Test Strip	CASPOFUNGIN	0.002 - 32	CAS	10 strips 30 strips 100 strips	921541 92154 921540
MIC Test Strip	FLUCONAZOLE	0.016 - 256	FLU	10 strips 30 strips 100 strips	921471 92147 921470
MIC Test Strip	FLUCYTOSINE	0.002 - 32	FC	10 strips 30 strips 100 strips	921491 92149 921490
MIC Test Strip	ITRACONAZOLE	0.002 - 32	ITC	10 strips 30 strips 100 strips	921481 92148 921480
MIC Test Strip	KETOCONAZOLE	0.002 - 32	KE	10 strips 30 strips 100 strips	921511 92151 921510
MIC Test Strip	MICAFUNGIN	0.002 - 32	MYC	10 strips 30 strips 100 strips	921821 92182 921820
MIC Test Strip	POSACONAZOLE	0.002 - 32	POS	10 strips 30 strips 100 strips	921521 92152 921520
MIC Test Strip	VORICONAZOLE	0.002 - 32	VO	10 strips 30 strips 100 strips	921501 92150 921500

MIC Test Strip for ANTIMYCOBACTERIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	ETHAMBUTOL	0.016 - 256	EB	10 strips 30 strips 100 strips	921701 92170 921700
MIC Test Strip	ETHIONAMIDE	0.016 - 256	ET	10 strips 30 strips 100 strips	921721 92172 921720
MIC Test Strip	ISONIAZIDE	0.016 - 256	IZ	10 strips 30 strips 100 strips	921711 92171 921710


MIC Test Strip ESBL for confirmation of Extended Spectrum Beta-Lactamase					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFEPIME/CEFEPIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.064-4	FEP/FEL	10 strips 30 strips 100 strips	921611 92161 921610
MIC Test Strip	CEFOTAXIME/CEFOTAXIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.016-1	CTX/CTL	10 strips 30 strips 100 strips	921601 92160 921600
MIC Test Strip	CEFTAZIDIME/CEFTAZIDIME+CLAVULANIC ACID (4 µg/mL)	0.5-32 / 0.064-4	CAZ/CAL	10 strips 30 strips 100 strips	921591 92159 921590


MIC Test Strip for KPC detection					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MEROPENEM / MEROPENEM + BORONIC ACID	0.125-8 / 0.032-2	MRP/MBO	10 strips 30 strips 100 strips	921671 92167 921670
MIC Test Strip	ERTAPENEM / ERTAPENEM + BORONIC ACID	0.125-8 / 0.032-2	ETP/EBO	10 strips 30 strips 100 strips	921681 92168 921680

MIC Test Strip MBL for detection of Metallo Beta-Lactamase					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	4-256 / 1-64	IMI/IMD	10 strips	921621
				30 strips	92162
				100 strips	921620
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	0.125-8 / 0.032-2	IMI/IMD	10 strips	921661
				30 strips	92166
				100 strips	921660
MIC Test Strip	MEROPENEM / MEROPENEM + EDTA	0.125-8 / 0.032-2	MRP/MRD	10 strips	921651
				30 strips	92165
				100 strips	921650


MIC Test Strip GRD for detection of Glycopeptide Resistance investigational use only					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	VANCOMYCIN / TEICoplanin	0.5-32 / 0.5-32	VA/TEC	10 strips	921631
				30 strips	92163
				100 strips	921630

MIC Test Strip for AmpC detection investigational use only					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFOTETAN / CEFOTETAN+CLOXACILLIN	0.5-32 / 0.5-32	CTT/CXT	10 strips	921641
				30 strips	92164
				100 strips	921640
MIC Test Strip	ERTAPENEM / ERTAPENEM + CLOXACILLIN	0.125-8 / 0.032-2	ETP/ECX	10 strips	921691
				30 strips	92169
				100 strips	921690


MTS Synergy Application System (patent pending)			
Description		Packaging	Ref.
MTS Synergy Applicator Platform		1 unit	96860
MTS Synergy Delivery Tool		10 disposable applicators	96870

Accessories			
Description		Packaging	Ref.
Automatic Rotation Plater 90 mm plates 14-110 rpm		1 unit	96795
Automatic Rotation Plater PRO (14-210 rpm)		1 unit	96796
Turntable extra wide (140 mm plates)		1 unit	96797
Wireless Foot Pedal		1 unit	96801
Steel Foot Pedal		1 unit	96798
Plastic Foot Pedal		1 unit	96799
STORAGE TUBE WITH SILICA GEL		10 tubes	96719
0.5 McFarland		1 tube	80400
1 McFarland		1 tube	80401
2 McFarland		1 tube	80402


Ready to use culture media			
DESCRIPTION	FORMAT	PACKAGING	REF.
MUELLER HINTON AGAR II for antimicrobial susceptibility testing of non-fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10031 10231
MUELLER HINTON AGAR II + SHEEP BLOOD 5% for antimicrobial susceptibility testing of fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10131 11231
MUELLER HINTON FASTIDIOUS AGAR (Horse blood 5% + 20 mg/L β-NAD) EUCAST recommended: for antimicrobial susceptibility testing of <i>Streptococcus</i> spp., <i>Haemophilus</i> spp., and some other fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10132 11132
MUELLER HINTON AGAR II + 2% NaCl for antimicrobial susceptibility testing of staphylococci with Oxacillin.	90 mm	20 plates	11206
CHROMATIC MH Chromogenic medium for the preliminary identification and susceptibility testing of bacteria directly from clinical and environmental specimens.	90 mm	20 plates	11618
BRUCELLA BLOOD AGAR with HEMIN AND VITAMIN K1 for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	10245
SCHADLER K AGAR (SHEEP BLOOD 5%) for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	11065
HAEMOPHILUS TEST AGAR for <i>Haemophilus</i> spp. susceptibility testing.	90 mm	20 plates	10080
MIDDLEBROOK 7H11 AGAR for antimicrobial susceptibility testing of mycobacteria.	90 mm	20 plates	10416
R.P.M.I. AGAR (2% glucose + MOPS) for antifungal susceptibility testing.	90 mm 140 mm	20 plates 10 plates	11509 10233

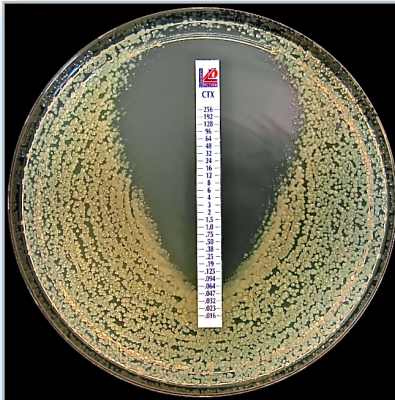


E. coli ATCC® 25922
MIC Test Strip CTX 0.016-256 µg/mL
M.I.C. = 0.094 µg/mL

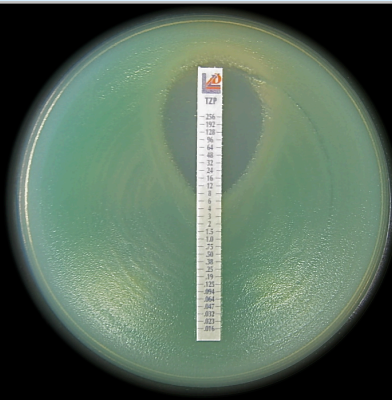


P. aeruginosa ATCC® 27853
MIC Test Strip TZP 0.016-256 µg/mL
M.I.C. = 8 µg/mL

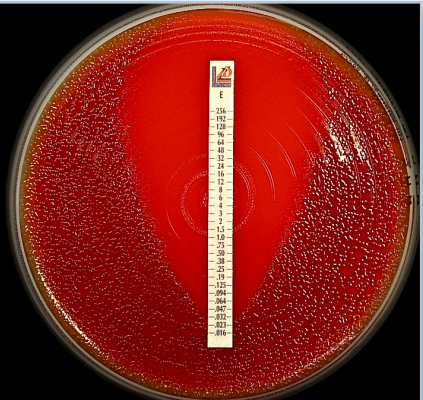


S. pneumoniae ATCC® 49619
MIC Test Strip E 0.016-256 µg/mL
M.I.C. = 0.032 µg/mL

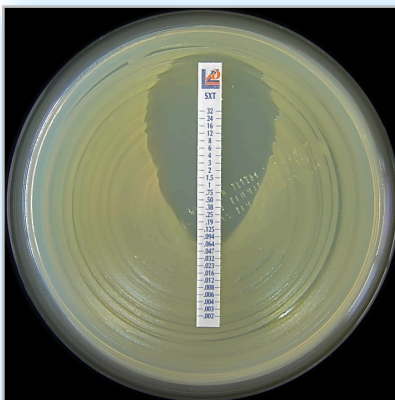


E. coli ATCC® 25922
MIC Test Strip SXT 0.002-32 µg/mL
M.I.C. = 0.094 µg/mL

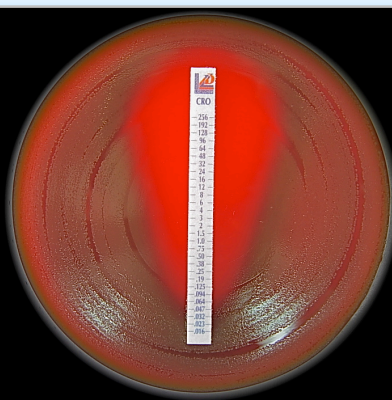


S. pneumoniae ATCC® 49619
MIC Test Strip CRO 0.016-256 µg/mL
M.I.C. = 0.064 µg/mL

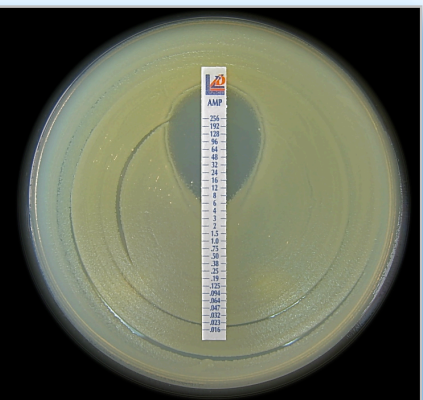


E. coli ATCC® 25922
MIC Test Strip AMP 0.016-256 µg/mL
M.I.C. = 8 µg/mL

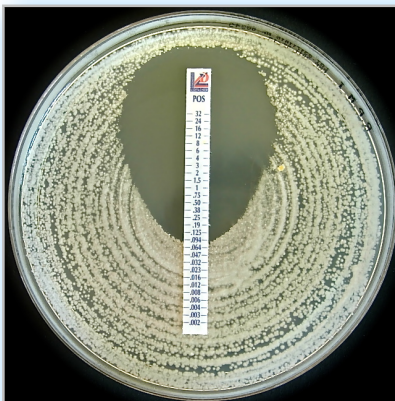


C. krusei ATCC® 6258
MIC Test Strip POS 0.002-32 µg/mL
M.I.C. = 0.094 µg/mL

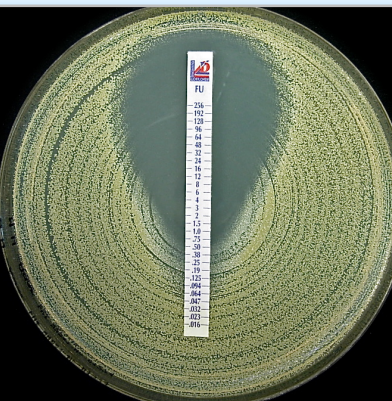


S. aureus ATCC® 29213
MIC Test Strip FU 0.016-256 µg/mL
M.I.C. = 0.064 µg/mL

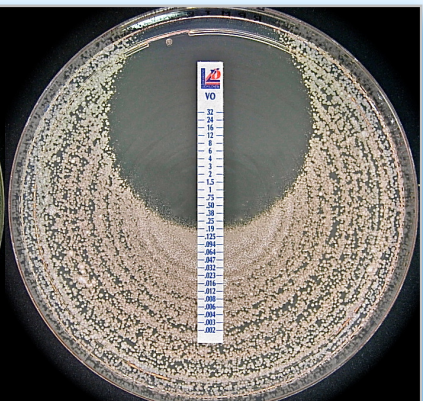


C. krusei
MIC Test Strip VO 0.002-32 µg/mL
M.I.C. = 0.25 µg/mL

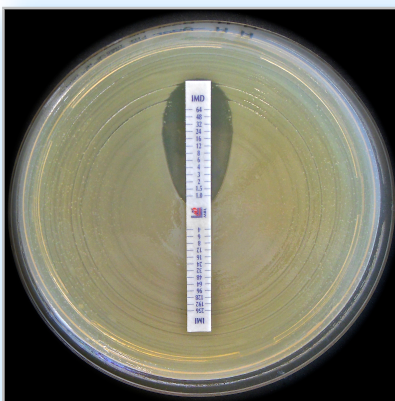

S. maltophilia ATCC® 13636
MIC Test Strip IMD/IMD 4-256/1-64 µg/mL
MBL+

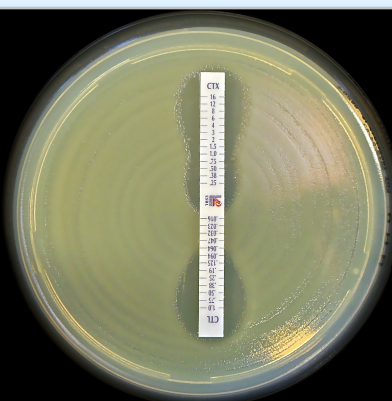

E. coli DSM 22311
MIC Test Strip CTX/CTL 0.25-1/0.016-1 µg/mL
ESBL+

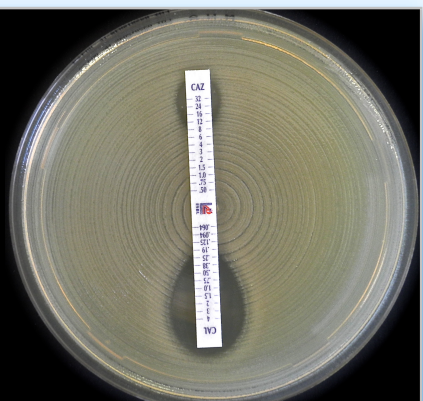

E. coli ATCC® 25922
MIC Test Strip CAZ/CAL 0.5-32/0.064-4 µg/mL
ESBL+

Contact your local dealer or Liofilchem headquarter for MIC Test Strip availability.


Algeria
Sarl Dimed
Tel +213 (0)21 75 53 80
Fax +213 (0)21 75 53 91
www.dimed-dz.com


Denmark
Biotech-IgG A/S
Tel +45 3538 0500
Fax +45 2538 7322
info@biotech-igg.com


Italy
Liofilchem
Tel +39 0858930745
Fax +39 0858930330
www.liofilchem.it


Saudi Arabia
Gamsco
Tel +966-3-8394085
Fax +966-3-8394084
drmalyousef@yahoo.com


Argentina
MedicaTec S.R.L.
Tel +54 11 4554.4600
Fax +54 11 4555.0416
www.medica-tec.com


Egypt
Media Medix Trade
Tel +20 22598417
Fax +20 24519256
mmediamedix@gmail.com


Japan
Alere Medical Co., Ltd.
Tel +81353267393
Fax +81353267177
www.alere.co.jp


Serbia
Uni-Chem
Tel +381113282986
Fax +381113281205
www.uni-chem.rs


Australia
Integrated Sciences Pty Ltd
Tel +61 2 9417 7866
Fax +61 2 9417 5066
www.integratedsci.com.au


Estonia
Borsa Eesti OÜ
Tel +3725050555
Fax +3726556158
borsa@borsa.ee


Lithuania
Biometrija
tel +37 052481824
fax +37 052400085
www.biometrija.lt


Singapore
Premier Diagnostics Pte Ltd
Tel +65 6220 3221
Fax +65 6220 9457
pdsingapore8@gmail.com


Belarus
Multi Lab Ltd.
Tel.: +375 17 309 17 20
Fax: +375 17 299 09 02
multilab@multilab.by


Finland
Immuno Diagnostic Oy
Tel +3583615370
Fax +35836822039
www.immunodiagnostic.fi


Malaysia
Premier Diagnostics Sdn Bhd
Tel +603-8024-0680
Fax +603-8024-0780
www.premierdiagnostics.com.my


Slovakia
Bio G spol. s r.o.
Tel +421245526381
Fax +421245526382
www.biog.sk


Benelux
ELITech Benelux B.V.B.A
Tel +32 (0)92820531
Fax +32 (0)92820532
c.notte@elitechgroup.com


France
i2a
Tel +33467504800
Fax +33467170906
www.i2a.info


New Zealand
Point View Laboratories Ltd
Tel +64 9 265 2884
Fax +64 9 265 0776
info@pointviewlaboratories.com


Slovenia
Mediline d.o.o.
Tel +38618308050
Fax +38618308070
www.mediline.si


Bulgaria
Elta 90M
Tel +35929839649
Fax +35929832211
www.elta90.com


Germany
bestbion dx GmbH
Tel +49 2234 987 95 0
Fax +49 2234 987 95 29
bernd.stammel@bestbion.com


Norway
Montebello Diagnostics AS
Tel 22 14 14 90
Fax 22 14 44 90
www.montebello.no


South Africa
Davies Diagnostics (PTY Ltd)
Tel +2711777600
Fax +27117876917
www.daviesdiagnostics.co.za


Canada
Alere Canada
Tel (613) 271-1144
Fax (613) 271-1148
www.alere.ca


Greece
Varelas S.A.
Tel +302105281920
Fax +302105220926
microbiology@varelas.gr



Pakistan
Imran Scientific Co
Tel 021-32210083
Fax 021-32210630
www.imransco.com


Spain
IZASA Distribuciones Técnicas, S.A.
Tel 902203090
Fax 902223366
www.izasa.es


Chile
Medica Tec de Chile S.A.
Tel +56 (02) 4261830
fax +56 (02) 4261835
mrojas@medica-tec.cl


Hungary
Frank Diagnosztika Kft.
Tel 388-3114, 367-2430
Fax 368-5721
www.frank-diagn.hu


Paraguay
GT Scientific S.A.
Tel 595 21 200134
Fax 595 21 221339
ventas@gtscientific.com.py


Sweden
Biotech-IgG AB
Tel +46 46 12 37 40
Fax +46 46 12 37 60
ak@biotech-igg.com


Croatia
Tamiko Instruments
Tel +38514683046
Fax +38514684811
www.tamiko.hr


Indonesia
PT Suryana Microbiology
Tel 62 21 345 2783
Fax 62 21 384 7232
najib@ptsuryana.com


Portugal
Izasa Portugal Distribuições Tecnicas Lda.
Tel +351 214161288
Fax +351 214176484
MIC Test Strip product range only


Thailand
Envimed Co., Ltd.
Tel 66-2-434-0391
Fax 66-2-435-5962
envimed@truemail.co.th


Cuba
CPM Cuba
Tel 2041432-2041433
Fax 2040987
microbiologia@cpm.co.cu


Iran
Pishro Imen Pajouh Co.Ltd.
Tel +98 21 44328639
Fax +98 21 44311632
pipco.ir@gmail.com


Romania
S.C. Sanimed International Impex SRL
Tel 021.420.54.94
Fax 021.420.54.93
www.sanimed.ro


Turkey
Kemiteks Kimyevi Mad. Tic. Ltd. Sti.
Tel +90 216 449 58 70
Fax +90 216 449 58 72
www.kemitekskimya.com.tr


Czech Republic
BioVendor
Tel +420549124123
Fax +420549211465
www.biovendor.cz


Ireland
Launch Diagnostics Ireland
Tel +3530719151122
Fax +3530719152277
desruddy@launchdiagnostics.ie


Russia
Agros-International, Zao
Tel +7 (812) 303-8234
Fax +7 (812) 303-8236
agros@agros-int.com


United Kingdom
Launch Diagnostics Ltd
Tel +44(0)1474874426
Fax +44(0)1474872388
enquiries@launchdiagnostics.com



Liofilchem®

Via Scozia zona ind.le, 64026 Roseto degli Abruzzi (Te) Italy

Tel. +39 0858930745 Fax +39 0858930330 www.liofilchem.net

liofilchem@liofilchem.net



ref. 6553003

Rev.15 / 08.08.2014